

CONTRAINDICATIONS POWER EYES MASSAGER

1. Before using, please consult a doctor if:

- You suffer from eye diseases such as glaucoma, leukoma and retinal detachment.
- You suffer from various weaknesses and fatigues, and also physical, mental or psychogenic disorders.
- You suffer from insult, hypertonia/hypotonia or heart diseases.
- You suffer from infectious diseases.
- You suffer from a cancer.
- You have had an eye surgery.
- You have a diagnosis of schizophrenia.
- You have open wounds, cuts, redness, infections, burns or bruises on the skin.
- You are pregnant, in the period of menstruation and incessant fever.
- You have a sensitive skin, angiectasia or were hospitalized.
- you have low temperature sensitivity, maximum temperature of the warming mode is 42°C.

2. Do not use the Massager when driving a car.

3. Do not disassemble the device yourself. If necessary, contact the service center.

4. Do not use, charge or store the device in hot or humid environment or in direct sunlight in order to avoid failure and malfunction.

5. Do not allow children and disabled people to use the device unattended.

6. Do not leave the massager under water or in rooms that provoke corrosion.

7. In case of malfunction stop using the device immediately.

8. Do not drop or hit the device.

9. Do not put heavy objects on the massager.

10. The massager is not a replacement for treatment or doctor's recommendations. If you are in a doubt please consult your doctor.

11. The massager shall not be used for diagnosis, treatment or for commercial purposes.

12. Do not use the massager during charging.